



Directions for Use:

1. Remove BioStick® from packaging
2. Remove battery cap and load with (2) AA batteries in correct position (batteries not included)
3. Replace battery cap
4. To turn device ON, push on/off switch upwards towards the "I" mark
5. When turned ON, device LED red light is lit and red light glows

Treatment:

1. Place the BioStick® on infected area with Red Light directly on infected area while device is lit and glowing
2. Place on infected area 1-2 minutes per session
3. Three (3) BioStick® sessions a day are recommended for best results
4. As skin heals, you may reduce BioStick® treatment to two sessions a day
5. After each use, clean Red Light tip of device with mild hand soap and washcloth
6. Do not place device directly under running water or immerse in water, soap, or any other liquid
7. Do not share use of BioStick® device. BioStick® is not intended for multiple users

Side Effects, Drug Interactions, Safety Hazards:


The red light omitted by the BioStick® is completely harmless. There are no known side effects for using the BioStick® three times a day.

CAUTION:

- Do not look directly at the light source
- Parental guidance required for children.



Problem/Malfunction	Possible Cause	Solution
Red light is not turning ON/BioStick® is NOT responding	1. Batteries are old/not charged	1. Change batteries - use fresh new batteries
	2. Batteries are placed incorrectly	2. Place batteries in correct order see diagram below
	3. Switch is Off	3. Turn switch On by pushing bottom up towards the I marking on the BioStick®
	4. Malfunction with product	4. If all problem/solutions above did not solve the malfunction, please contact your local vendor.
Green light turns ON	1. Batteries are weak	1. Batteries must be replaced with fresh/new batteries

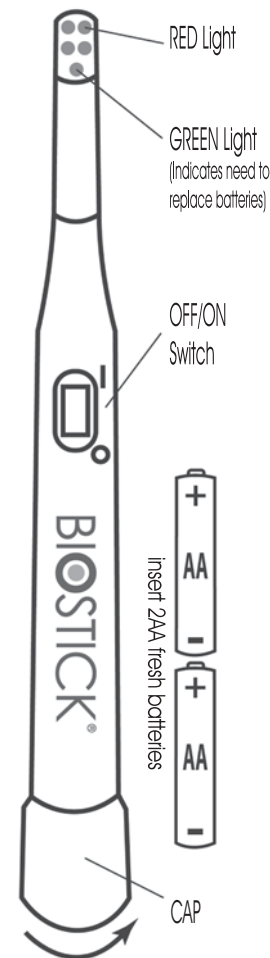

100% Satisfaction Guarantee

Due to world-wide distribution of BioStick®, various customized-warranty may be provided from country to country, please contact your local vendor for warranty detail at your best interest.

Light Treatment	Narrow band 630nm-660nm light
Illumination Power	18mW (typical)
Turn On	Turn the bottom cap clockwise
Power Source	2 AA Alkaline Batteries (not included)
Current Drain	90mA
Battery Life	Approximately 1000 Sessions of 1 minute
Low Battery Indication	Green LED lights up when battery is low
Unit Weight	75g with battery, 28g without battery
Unit Size	Lenght: 200mm, Diameter: 19mm

Authorized Representative In Europe: MEDES LIMITED,
 9 Beaumont Gate, Shenley Hill, Radlett, Herts, WD7 7AR,
 England, Tel / Fax: + 44 19 23859810
 Certified for: ISO 9001 (2000), MDD93/42 (1993),
 ISO 13485 (2003) Cert. No. 499/499CE

CE 0473



Made by SyroLight Ltd
 9 Hadror St. Jaffa-Tel-Aviv, Israel
 Tel: 972-3-5332274
 Fax: 972-3-5332275
 www.thebiostick.com

